

Baby Weighing Scale

AV-WS-2

Keys' function

- **ON/TARE:** Press the key to power on the scale and start up weighing process
- **MODE:** Weighing units exchange key. Press the key to select kg and lb in turn
- **MEN:** While the scale is in "0.00" situation, you can press this key to recall the former one weighing value saved in the memory and the suitable indicator is lighted

Operation

- After the batteries placed, press "ON/TARE" key to start the scale until "0.00" appearance
- Gently place baby on the pan, weighing record will be flashed on the window after a few seconds, the scale will turn off automatically within 5 minutes if no operation again.
- If see about the former saved weighing value, please press the "MEN" key while the scale is in "0.00" position. The scale will save optional value during each weighing operation

Cautions

- You must replace batteries immediately when window display "Loo"
- Keep the scale clean to ensure the normal working
- Over load (heavier than 20.5kg) is strictly prohibited



Aveus Instruments Sp. z o.o.

17/19/133 Domaniewska Street
02-672 Warsaw - Poland

info@aveusinstruments.com.pl
www.aveusinstruments.com.pl